

**HEMA-SCREEN SPECIFIC
IMMUNOLOGICAL OCCULT BLOOD TEST**

PATIENT COLLECTION INSTRUCTIONS:

Patients that use the DEVEL-A-TAB Sampler will collect specimens at home over the course of two consecutive bowel movements. Using a ballpoint pen, write your name, date of birth and date of each bowel movement on card. Prior to defecation: Flush the toilet to clear bowl. For ease of collecting specimen, place tissue provided onto water in bowl. Lift front flap on test slide. After defecating into bowl, using applicator stick, stab stool in 4 different sites of the bowel movement and apply to circle # 1. Close flap. Discard stick in trash. **DO NOT FLUSH APPLICATOR STICK.** Repeat the same procedure for the next bowel movement and apply to circle #2. Place the card into the provided mailing envelope, seal by removing tape strip and fold as indicated. Mail or return to the laboratory.

IT IS AGAINST POSTAL REGULATIONS TO MAIL SPECIMEN IN A STANDARD PAPER ENVELOPE.

***No dietary restriction is required for this occult blood testing procedure. The use of only two samples of feces is required for testing.